



Battery Status

Press and release button to show battery power status.

Indicator Light	Charge Level
Solid Green	51%-100%
Flashing Green	21%-50%
Solid Red	11%-20%
Flashing Red	1%-10%



Change Footwear

■ **Change shoes:** Put on different shoes, stand with toes even and feet shoulder width apart. Press and hold Change Footwear button on front of ankle until white light starts to flash. Light will stop flashing when complete. If unsuccessful, you will see alternating flash of white change footwear and red battery button. Press Change Footwear button to stop flashing light.



Bluetooth

- To pair ankle to App, press button once when prompted by App.
- If blue light is blinking, press and hold button until light turns off. This will allow the user to connect to Kinnex App.



Storage Mode

- **To put into storage mode:** Press and Hold Bluetooth and Change Shoes buttons for 6 seconds. All 3 lights will FLASH on and off simultaneously, after lightsturn off, Kinnex will enter storage mode
- **To take out of storage mode:** Plug Kinnex into charging source.

Manual Lock

- **To Activate:** Double press the Change Footwear Button located on the key pad or by activate within the Kinnex App. The Change Footwear Button white indicator light will turn on while the ankle is locked. The Manual Lock can lock in any position throughout the available ROM.
- **To Disengage:** Press the Change Footwear Button twice quickly again or turn off in the App. Once disengaged, the Change Footwear Button light will turn off and the ankle will return to normal function.

IMPORTANT: The Manual Lock CANNOT be activated when battery is dead.